

# ASM


User Instructions  
ASM Suffolk Square Section  
Adjustable Chair Code 01304.

## Medicare

Picow Farm Road Ind Est  
Runcorn. Cheshire.  
WA7 4UG.  
Tel: 01928 574301.  
Fax: 01928 575130.  
e-mail: [sales@asmmedicare.com](mailto:sales@asmmedicare.com)



## ASM Suffolk Square Section Adjustable Chair 01304.

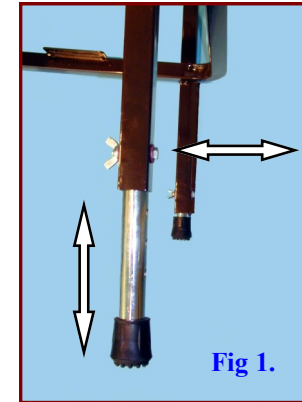

On receipt of your new ASM Suffolk Square Section Adjustable Chair, please check for signs of damage, report any defects to ASM immediately. The ASM Suffolk Chair comes fully assembled and ready for use. Various options are available, please ring for details. The intended use of this chair is to provide an adjustable seat support around the home or care facility. The chair is height adjustable, to adjust please follow the instruction below.

- 1} Place the chair on it's side and remove the wing nuts & bolts from the two legs. See Fig 1.
- 2} Slide the adjustable legs to the required position, ensuring that both legs are adjusted equally.
- 3} Refit the bolts, you may have to twist the adjustable legs slightly, and tighten the wing nuts.
- 4} Turn the chair over & repeat for the other two adjustable legs.
- 5} Place the chair upright on it's feet on a level surface and check that it is stable & does not rock.  
If it does rock readjust the legs.
- 6} Periodic checks of all screws & fittings is recommended, replace if necessary.
- 7} Clean the ASM Suffolk Chair, a mild soap and / or disinfectant may be used. After cleaning dry with a soft clean cloth.

The **Maximum User Weight** limit for this product is: 159kg / 25st.

**Note:** This product must only be used for it's intended purpose.  
For details of all our products please visit our web site at;  
[www.asmmedicare.com](http://www.asmmedicare.com)



October 2008.

*Better By Design*