

# ASM



## IMPORTANT NOTICE!

Please read & fully understand before using or allowing others to do so.

# Medicare

Picow Farm Road Ind Est  
Runcorn, Cheshire.  
WA7 4UG.

Tel: 01928 574301.

Fax: 01928 575130.

e-mail: [sales@asmmedicare.com](mailto:sales@asmmedicare.com)

## Stackwell Commode.

Code No: 02122 Fixed Height With  
Detachable Upholstery.



Fig 1.



On receipt of your new ASM Stackwell Commode please check for signs of damage, report any defects to ASM immediately. The ASM Stackwell Commode comes fully assembled and ready for use once the commode pot is fitted. To fit the commode pot please follow the instructions below;

- 1) Remove all packaging and lift the toilet seat, fit the commode pot so that it rests on the two support brackets, ensuring that it sits centrally and is fully supported.
- 2) To remove the pot, simply lift the toilet seat once more and use the attached moulded handle to lift. (See Fig 1).
- 3) Please ensure the pot does not become over full as this may lead to spillage.
- 4) Periodic checks of all screws & fittings is recommended, replace if required.
- 5) To clean the Stackwell Commode, a mild proprietary soap and / or disinfectant may be used. After cleaning dry with a soft clean cloth.

**Maximum User Weight for this product is: 102kg / 16st.**

A heavy duty model is available with a 190kg / 30st MUW. Just add HD to the end of the product code number.

**NOTE:** Use only minimum force when sitting. Do not “drop down onto the seat”.

If further support is required when sitting / standing please visit our web site for details of all our products.

This product must only be used for it's intended purpose.

*Better By Design*

