

ASM

Medicare

Picow Farm Road Ind Estate
Runcorn. Cheshire. WA7 4UG.
Tel: 01928 574301
Fax: 01928 575130
E-mail: sales@asmmedicare.com

User
Instructions
ASM Avon
Shower Chair.

Avon Commode 01099.



On receipt of your new ASM Avon Commode please check the contents for signs of damage, report any defects to ASM immediately. The Avon comes fully assembled and ready for use & is available either chrome plated or coated in a white plastic finish. The Avon is supplied with detachable arms as standard, these are removed by simply lifting them out of their location tubes, this facility is particularly useful for side transfer. Do Not Try To Lift The Commode With The Arms!

The commode pot slides into two brackets underneath the seat and is supplied with a lid. **DO NOT** allow the pot to become over full, as this may lead to spillages.

The castors MUST be locked in the Forward position at ALL times when the commode is stationary. You must ONLY lean forward when the castor locks are on and in the forward position, as shown in Fig 1. The chair may over balance if not!

To remove the optional footrests, lift upwards, tilt side ways and remove. Standing on this chair or the footrests is **VERY** dangerous and could lead to personal injury. The optional overlay panel is provided as a cover **ONLY** and must **NOT** be used as a seat.

Clean the chair & pot using a mild proprietary soap and / or disinfectant, diluted bleach can also be used but **MUST** be completely washed off immediately. Dry with a soft clean cloth. Check **ALL** screws & fittings regularly to ensure correct operation.

The intended use of this product is to provide the user with a mobile commode facility, which can be used over an existing toilet without modification.

The Maximum User Weight for this product is 190kg / 30st.

NOTE:- These products are only to be used for their intended purpose.
For details of all our products please visit our web site at www.asmmedicare.com



Please read & fully understand before using or allowing other to do so.



Nov 2009.