


ASM

Medicare

Picow Farm Road Ind Estate
Runcorn. Cheshire. WA7 4UG.
Tel: 01928 574301
Fax: 01928 575130
E-mail: sales@asmmedicare.com



Cosby Toilet Frame 07060A

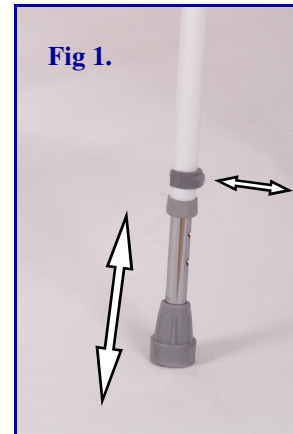

On receipt of your new ASM Cosby Toilet Frame please check the contents for signs of damage, report any defects to ASM immediately. The Windsor comes fully assembled and ready for use.
The Windsor is height adjustable, to adjust please follow the instructions below.

- 1} Lay the frame on it's side and remove the plastic clips from 2 legs. See Fig 1.
- 2} Slide the internal legs to the required position ensuring that both legs are adjusted equally.
- 3} Replace the plastic clips, you may have to twist the leg slightly to allow the steel pin to locate completely, it is "Very Important" that the clips are fully engaged.
- 4} Turn the frame over and repeat for the other 2 remaining legs.
- 5} Place the Cosby Toilet Frame back on it's feet on a level surface and check that it is stable and does not rock, if it does, readjust until it is stable. The plastic seat should *not* touch the toilet pane when in use, as this could cause damage to the sleeve.
- 6} Periodic checks of *all* screws and fittings is recommended, replace if required.
- 7} The clip-on plastic seat is removable for ease of cleaning and can be cleaned in the same manner as the frame, by using a mild proprietary soap and / or disinfectant. Dry with a soft clean cloth.

The intended use of this product is to provide the user with a height adjustable toilet seat, which can be used over an existing toilet without modification.

The Maximum User Weight for this product is 190kg / 30st.

NOTE:- These products are only to be used for their intended purpose.
For details of all our products please visit our web site at www.asmmedicare.com

User
Instructions
ASM Cosby
Toilet Frame.

