

ASM

Medicare

Picow Farm Road Ind Estate
Runcorn, Cheshire. WA7 4UG.
Tel: 01928 574301
Fax: 01928 575130
E-mail: sales@asmmedicare.com



**CUMBRIA HEAVY DUTY
TOILET SURROUND.
Standard Code: 02047HD
Floor Fixed Code: 02047HDFP
{not shown}**



User Instructions:
ASM Cumbria Heavy Duty
Toilet Surround.

On receipt of your new ASM Cumbria Toilet Surround Heavy Duty please check the contents for signs of damage, report any defects to **ASM Medicare Ltd** immediately. The Cumbria comes fully assembled and ready for use.

- 1} Place the frame around the toilet, ensuring that the frame is level and does not rock or wobble.
- 2} If the frame does wobble, either re-site the frame, ensure that the floor is level or purchase the standard height adjustable model (Code No: 02047 190kg / 30st).
- 3} A floor fixed model is also available (Code No: 02047HDFP). Please ensure that the appropriate screws and plugs as required by your type of floor are used. These are not supplied and we also recommend that fitting is carried out by a competent person or tradesman.
- 4} Periodic checks of *all* screws and fittings is recommended, re-tighten or replace if required.
- 5} Clean the Cumbria by using a mild proprietary soap and / or disinfectant. Dry with a soft clean cloth.

The intended use of this product is to provide the user with a toileting support aid to fit around a standard toilet, which can easily be removed when not required. It is not designed as a walking aid and should not be used as such.

M.U.W = 255kg / 40st.

NOTE:- These products are only to be used for their intended purpose.
For details of all our products please visit our web site at www.asmmedicare.com



October 2005