

ASM


Medicare

ASM
Bedstick Range.
User
Instructions.

Picow Farm Road Ind Est
Runcorn, Cheshire.
WA7 4UG.
Tel: 01928 574301.
Fax: 01928 575130.
e-mail: sales@asmmedicare.com



Fig 1.

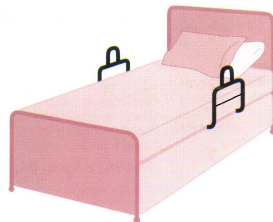

Please Note:

For older models of this product i.e. without the safety entrapment bar or for children, Bedstick mesh covers are available. These can be fitted easily to any ASM Bedstick.
Please call our sales team for details.



ASM Bedstick Range.


Code No's:

02050A ~ Loop Grips.

02050B ~ Straight Grips.

02050AB ~ Combination of Grips.

**MFSA or MFSB To Suit Grip ~
Mesh Covers for Children.**

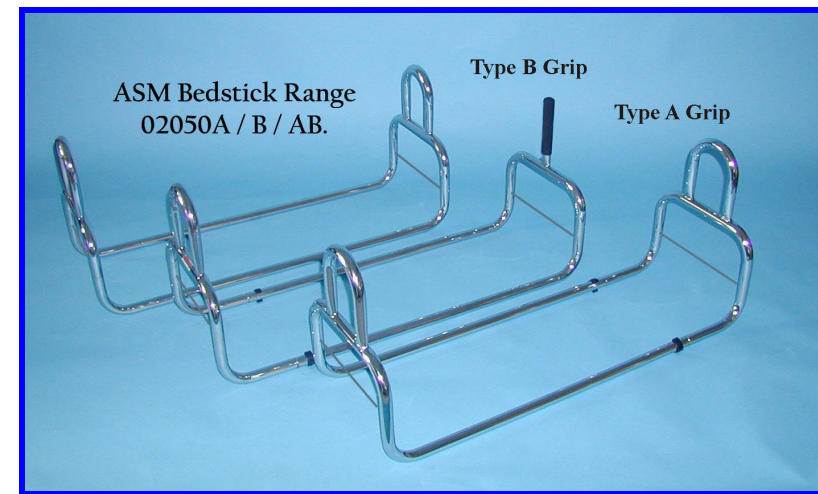

On receipt of your new ASM Bedstick please check the for signs of damage, report any defects to ASM immediately.

The Bedsticks come fully assembled & ready for use.

These products are available in fixed width of 762mm {30"} or adjustable width which will fit beds from 944mm {36"} to 1372mm {56"}.

They are also available with different types of grip as illustrated, please specify on order.
To fit & adjust please follow the instructions below.

- 1} Place the Bedstick upside down on the mattress and remove the two plastic clips.
- 2} Slide the Bedstick apart to the desired size and line up the nearest holes in the inner & outer parts.
- 3} Refit the plastic clips.
- 4} At this stage it is necessary to either remove the mattress or slide the Bedstick underneath from one side.
We recommend that you seek assistance to remove it to avoid any personal injury.
- 5} Place the Bedstick on top of the bed base ensuring that is the correct way up, (see Fig 1) replace the mattress.
- 6} If a child is to use the Bedstick you must ensure that there is NO risk of the users head or body sliding between the mattress and the Bedstick, especially where a thinner type of mattress is used. Seek advise if in any doubt.
- 7} Clean the Bedsticks with a mild propriety soap and or disinfectant may be used, dry after cleaning with a soft cloth.

Note: It is intended that these products should be used as a support aid for a person getting into & out of bed, and should only to be used for it's intended purpose.

Maximum User Weight for these products are;

02050A ~ 152kg (24st) / 02050B ~ 102kg (16st) / 02050AB ~ 102kg (16st).

For details of all our other products please visit our web site at www.asmmedicare.com

Better By Design