

ASM

User
Instructions
ASM Wren
Kitchen /
Shower Stool
Range.



IMPORTANT NOTICE!

Please read & fully
understand before
using or allowing
others to do so.



Medicare

Picow Farm Road Ind Estate
Runcorn, Cheshire. WA7 4UG.

Tel: 01928 574301

Fax: 01928 575130

E-mail: sales@asmmedicare.com

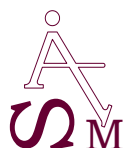


Fig 1.



TRAYS:

The plastic trays simply clip on or off for ease of cleaning. The wooden trays are not designed to be removed for cleaning. Replacement trays are available upon request.

Kingfisher Trolley Range.

Codes:

02129 ~ White Frame & White Trays.

02130 ~ Brown Frame & Cream Trays.

02130BU ~ Burgundy Frame & Grey Trays.

Add PH after product code if rear pushing handle is required, for single hand use.



On receipt of your new ASM Kingfisher Trolley please check for signs of damage, report any defects to **ASM Medicare Ltd** immediately.

The ASM Kingfisher Trolley come fully assembled and ready for use.

The trolleys are height adjustable, to adjust please follow the instructions below.

- 1} Place the trolley on it's side and remove the plastic clips from 2 legs. See Fig 1.
- 2} Slide the adjustable internal legs to the required position ensuring that both legs are adjusted equally. See Fig 1.
- 3} Replace the 2 plastic clips, ensuring that they are **fully engaged**. You may have to twist the leg slightly to locate the steel pin correctly. When engaged correctly the end of the steel pin should be flush with the trolley leg.
- 4} Turn the trolley over & repeat for the other two adjustable legs.
- 5} Place the trolley upright on it's castors, on a level surface & check that it is stable & does not rock, if it does readjust the legs until it is stable.
- 6} Periodic checks of all screws and fittings is recommended, replace if required.
- 7} To clean the Kingfisher Trolley, a mild proprietary soap and / or disinfectant may be used. After cleaning dry with a soft clean cloth. Alternatively these products may be cleaned using an automatic system such as the "Autoclave".

NOTE:- These products must only be used for their intended purpose.

The trolleys are designed for transporting household items, such as drinks, meals etc. They are **NOT** designed as a walking aid & should not be used as such. Personal injury could occur if not used correctly.

For details of all our products please visit our web site at www.asmmedicare.com