

ASM

Medicare

Picow Farm Road Ind Estate
Runcorn. Cheshire. WA7 4UG.

Tel: 01928 574301

Fax: 01928 575130

E-mail: sales@asmmedicare.com



Fig 1.


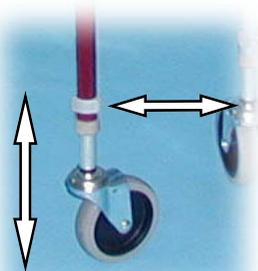


Fig 2.

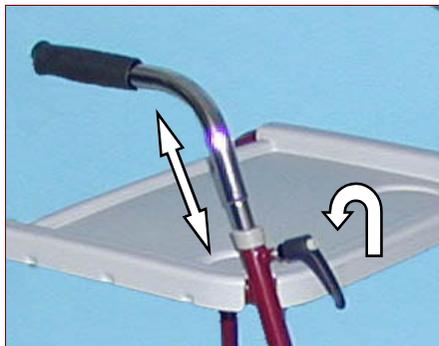

Kingfisher Walker Trolley Range.

Code No's:

- 02133 ~ Burgundy Frame & Grey Wooden Trays.
- 02134 ~ Burgundy Frame & Grey Plastic Trays.
- 02135 ~ Green Frame & Grey Wooden Trays.
- 02136 ~ Green Frame & Grey Plastic Trays.
- 02137 ~ Blue Frame & Grey Wooden Trays.
- 02138 ~ Blue Frame & Grey Plastic Trays.
- 02139 ~ White Frame & White Plastic Trays.



On receipt of your new ASM Kingfisher Walker Trolley please check for signs of damage, report any defects to **ASM Medicare Ltd** immediately.

The ASM Kingfisher Walker Trolley comes fully assembled and ready for use.

The trolley legs & handles are height adjustable, to adjust please follow the instructions below.

- 1} Place the trolley on it's side and remove the plastic clips from 2 legs. See Fig 1.
- 2} Slide the adjustable internal legs to the required position ensuring that both legs are adjusted equally. See Fig 1.
- 3} Replace the 2 plastic clips, ensuring that they are **fully engaged**. You may have to twist the leg slightly to locate the steel pin correctly. When engaged correctly the end of the steel pin should be flush with the trolley leg. Turn the trolley over & repeat for the other two adjustable legs.
- 4} Place the trolley upright on it's castors, on a level surface & check that it is stable & does not rock, if it does readjust the legs until it is stable.
- 5} To Adjust the handle height, loosen the black handle at the front of the trolley, (turning it anti-clockwise) set to the desired height and re-tighten the black handle. See Fig 2. Repeat for the other handle.
- 6} Periodic checks of all screws and fittings is recommended, replace if required.
- 7} To clean the Kingfisher Trolley, a mild proprietary soap and / or disinfectant may be used.
After cleaning dry with a soft clean cloth.

Replacement trays, clips, handles & castors are available on request.

M.U.W ~ 102kg / 16st. {Frame not trays}.

NOTE:- These products must only be used for their intended purpose.

The trolleys are designed for transporting household items, such as drinks, meals etc.

The trolley's are designed to be used as a walking aid for indoor use **ONLY**.

For details of all our products please visit our web site at www.asmmedicare.com

Better By Design

User
Instructions
ASM Kingfisher
Walker Trolley
Range.

IMPORTANT NOTICE

Please read & fully
understand before
using or allowing
others to do so.

