



ASM

Medicare

Picow Farm Road Ind Estate
Runcorn, Cheshire. WA7 4UG.
Tel: 01928 574301
Fax: 01928 575130
E-mail: sales@asmmedicare.com

**EURO BEDSTICK.
Code No: 02051.**

Fig 1

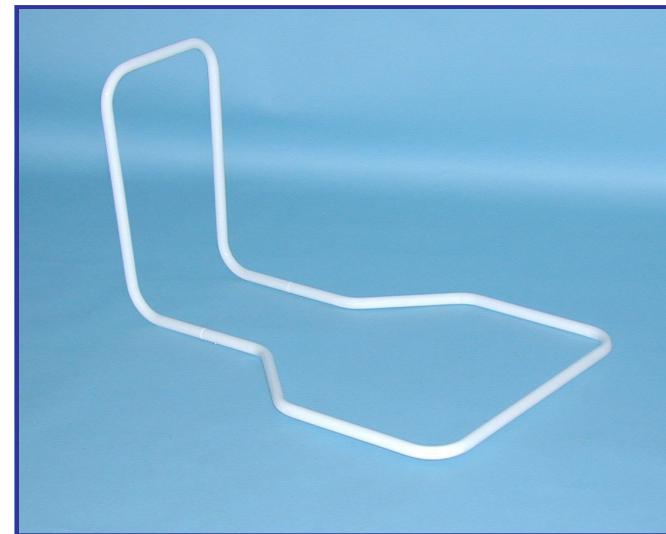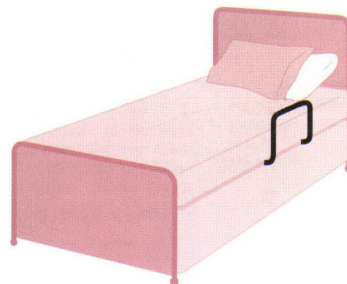

**User Instructions:
ASM Euro Bedstick.**

On receipt of your new ASM Euro Bedstick please check for signs of damage, report any defects to **ASM Medicare Ltd** immediately. The Euro Bedstick comes fully assembled and ready for use. To fit your Bedstick please follow the instructions below.

- 1} Lay the large section of the Bedstick onto the bed base underneath the mattress. See Fig 1.
- 2} We recommend that you seek assistance when lifting the mattress to avoid any personal injury.
- 3} Replace the mattress.
- 4} If a child is the intended user of the Bedstick you **MUST** ensure that there is **NO** risk of the user's head or body sliding inside the handle of the Bedstick . Especially where a thinner type of mattress is used.

IF IN ANY DOUBT SEEK ADVICE!

Mesh covers are available for this product in order to reduce the risk of entrapment, please call for details.

- 5} Clean using a damp cloth and a proprietary soap / disinfectant.

The intended use of this product is to provide the user with a support aid, whilst getting in & out of bed.

M.U.W = 102kg / 16st.

NOTE:- These products are only to be used for their intended purpose.
For details of all our products please visit our web site at www.asmmedicare.com



June 2006