

ASM

ASM Denby Monkey Lifting Pole User Instructions.

Medicare

Picow Farm Road Ind Est
Runcorn. Cheshire.
WA7 4UG.

Tel: 01928 574301.

Fax: 01928 575130.

e-mail: sales@asmmedicare.com

ASM Denby Monkey Lifting Pole 02180.

Parts List:

- > Pole.
- > Back Frame.
- > Base Frame.
- > Connector & Lifting Strap.



On receipt of your new ASM Denby Monkey Lifting Pole, please check the contents for signs of damage, report any defects to ASM immediately. The monkey pole comes flat packed, please follow the instructions below carefully.

- 1) Place the base frame on the floor.
- 2) Slide the back frame onto the base frame uprights.
- 3) Slide the monkey pole into the back frame ensuring that it fits into both top & bottom brackets.
- 4) Tighten the hand wheel on the back frame.
- 5) Slide the strap connector over the end of the pole making sure that it locates into the required adjustment hole.
- 6) Clean using warm water and a mild proprietary soap or disinfectant, dry with a soft cloth or towel.

The monkey pole must be correctly placed to the bed or chair with which it is to be used, the monkey pole base **MUST BE**

UNDERNEATH the bed or chair to provide correct operation.

If in any doubt seek advice as failure to do this correctly could lead to personal injury.

The intended use of this product is to provide the user with a lifting aid to assist in movement whilst sitting or in bed.

Maximum User Weight: 102kg / 16st.

Note: This product will only support a proportion of the user's body weight equal to 50%.

This product is **ONLY** to be used for it's intended purpose.

For details of all our products please visit our web site at www.asmmedicare.com

Better By Design



**IMPORTANT
NOTICE:**
Please read &
fully understand
before using or
allowing others
to do so.

