

User Instructions ASM Langley Footstool.

Medicare

Picow Farm Road Ind Estate Runcorn. Cheshire. WA7 4UG. Tel: 01928 574301

Fax: 01928 575130

E-mail: sales@asmmedicare.com

Langley Footstool 01253.

Also Available with Castors: Code No: 01253C





On receipt of your new ASM Langley Footstool please check the contents for signs of damage, report any defects to **ASM Medicare Ltd** immediately. The Langley comes fully assembled and ready for use.

The Langley comes supplied with or without castors, is height adjustable. Both models come in easy to clean vinyl with chromed legs and frame.

The design of this footstool allows various positions to be adopted for maximum comfort of the user, particularly useful for anybody suffering from an injury to the leg or foot. The adjustment of the stool is a very simple operation, please follow the instructions below;

- 1} Turn the footstool upside down and lay on the floor on it's back.
- 2} On the inside of each leg is a spring loaded location pin, press the pin in with your thumb and slide the leg to the required position.

 The pin will self locate in the next hole, repeat if further adjustment is required.
 - 3} The footstool does not have to be adjusted so that it is level and can therefore be used at convenient angles to suit the user.

It is "Very Important" that both the pins are fully engaged before use.

- 4} Periodic checks of all screws and fittings is recommended, replace if required.
- 5} Clean the footstool by using a mild proprietary soap and / or disinfectant. Dry after cleaning with a soft clean cloth.

The intended use of this product is to provide the user with a height adjustable foot support aid & must only be used as such. Do not use as a chair or step stool as this could result in personal injury! M.U.W = 190 kg / 30 st.

NOTE:- These products are only to be used for their intended purpose.

For details of all our products please visit our web site at www.asmmedicare.com



IMPORTANT NOTICE!

Please read & fully understand before using or allowing others to do so.

