

ASM Medicare

Taunton High Back Chair. Code No: 01308.




CODE 01308

IMPORTANT NOTICE!
Please Read And Understand
Fully Before Use Or
Allowing Others To Use.

Picow Farm Road
Runcorn, Cheshire
WA7 4UG
Tel: 01928 574301
Fax: 01928 575130
Email:- sales@asmmedicare.com

On receipt of your new ASM Taunton High Back Chair please check for signs of damage, report any defects to ASM immediately. The ASM Taunton High Back Chair comes fully assembled and ready for use. The arm chair is height adjustable, to adjust please follow the instructions below.

Fig 1

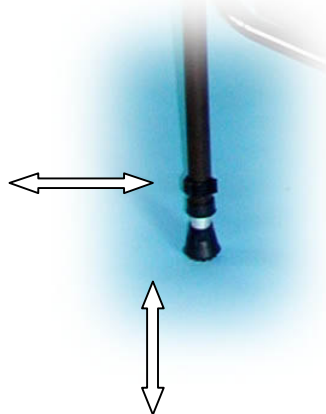

- 1) Place the chair on it's side and remove the plastic clips from two legs. See Fig 1.
- 2) Slide the adjustable internal legs to required position ensuring that both legs have been set at the same height.
- 3) Refit the 2 plastic clips, ensuring that they are **fully engaged**. You may have to twist the legs slightly to locate the steel pins correctly.
- 4) Turn the chair over & repeat for the other two adjustable legs.
- 5) Place the chair upright on it's feet on a level surface & check that it is stable & does not rock, if it does readjust the legs until it is stable.
- 6) Periodic checks of all screws & fittings is recommended, replace if required.
- 7) To clean the Taunton Chair, a mild soap and / or disinfectant may be used. After cleaning dry with a soft clean cloth.

Maximum User Weight for this product is;
152kg (24st).

Note: This product must only be used for it's intended purpose.
For details of all our products please visit our web site at
www.asmmedicare.com

ASM TAUNTON HIGH BACK CHAIR USER INSTRUCTIONS

